

## Log In

Go to <http://www.corestudycast.com/>.

Click Client Login in the top right.

The first time you log in, you will be required to change your password and acknowledge the HIPAA policy.



## Studies Page

Each time you log in, the Studies page is displayed. This page lists all studies you are authorized to view.

Study ID	Date of Service	Study Type	Status	Patient Name	DOB	Patient
60	2021-04-06 08:58:44	Obstetric	Preliminary	Brooks, Kaylin	1992-11-14	MHRC-04062
56	2021-04-06 11:49:17	Obstetric	New	Fiore, Cynthia	1984-02-09	MHRC-04062
45	2021-04-06 10:50:56	Obstetric	New	Carter, Alicia	1987-09-17	MHRC-04062
67	2021-04-06 09:54:41	Obstetric	New	Acharya, Priyanka	1989-12-14	MHRC-04062

### Studies are color coded by status



**New** Uploaded but not yet available for the reading physician.



**Preliminary** Ready for the reading physician to review.



**Final** Approved by the reading physician, and the final report has been generated.

### The Study ID column displays three clickable icons



Lists the number of images in the study. Click to view the images.



Click to open the worksheet.



Click to view the report (Icon only appears if a report has been created).

Study ID	Date of Service	Study Type	Status	Patient Name	DOB	Patient
84		A-V Fistula				
		AAA				
		Abdomen				
		Abdomen and Pelvis XRay				
		Abdomen XRay				
		Abdominal (vet)				
		Abdominal Aorta				
		ABI				
		Angiography				
		Ankle XRay				
		ANS HRV				

### To filter the list of studies

1. Click **Select Criteria** and select an item from the list.
2. Enter the requested information.
3. Click the **Search** button.

To clear all search criteria and return to the complete list of studies, open the drop-down menu on the **Search** button and select **Reset**.



## Complete the Worksheet

Measurements in the worksheet are automatically populated from the modality.


	Right Breast	Left Breast	Report Images	Report Attachments	Report Recipients			
<b>Masses:</b>	Lat (cm)	Trans (cm)	AP (cm)	Location	Echotexture	Area	Appearance	
Mass 1	0.7	X 0.9	X 1.1	2 o'clock	Homogeneous			
Mass 2		X	X		Heterogeneous			
Mass 3		X	X		Hyperechoic			
					Hypoechoic			
<b>Cysts:</b>	Lat (cm)	Trans (cm)	AP (cm)	Location	Type	Area	Appearance	
Cyst 1		X	X	None				
Cyst 2		X	X					
Cyst 3		X	X					

You will need to enter some additional information and select observations from drop-down lists.

To edit text, click the text and begin typing.

To enter text in an empty field, click the dots (...) and begin typing.

**Some text fields have a list of items you can choose.** To look up a specific item, begin typing. The list appears and narrows as you type. To see the entire list, click in the field and press the space bar twice.

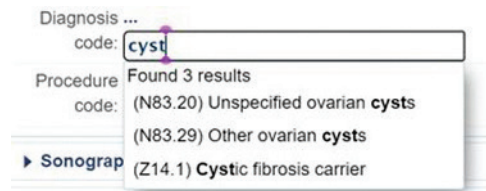

**To generate findings statements,** click the Generate All Findings button.

When you are ready for the reading physician to review the study, click the Set Preliminary button in the bottom right. The study is routed to the physician.

Access the Studycast system from any Internet-connected device.

For more information, click Help on any page, view the Tutorial Videos, or contact Support.

**Phone** 866.209.3393 x3

**Email** support@corestudycast.com



www.corestudycast.com

Copyright © 2022 Core Sound Imaging, Inc.  
LBL-72-03-207-B Effective 02/27/22